

ADVANCE CARE *IP*

Foldable Out-of-Bed Floor Mat

The Advance Care Foldable Out-of-Bed Floor Mat is a pressure sensitive device which sends off an alert the moment a resident steps out of their bed.

Composed of textured, non-slip material, the mat is designed to be both warm and comfortable underfoot. Aesthetically speaking, the mat is provided in a natural beige hue, ensuring that it is totally unobtrusive - simply becoming part of the room's decor.

With dimensions of 1200mm x 600mm, staff need not be concerned that a resident will exit their bed un-noticed; the length of the mat parallels with the length of a standard hospital bed.

The mat is water resistant, washable and durable, ensuring a lengthy product life.

A crucial addition in any facility's Dementia Care Management plan, when integrated with any Advance Care IP Nurse Call System, staff and management can specifically configure the Foldable Out-of-Bed Floor Mat with a myriad of operation techniques. The mat may also be configured to turn the lights on when the resident arises - a useful safety feature for those who are less mobile.

When the Foldable Out-of-Bed Floor Mat receives pressure, a high visibility reassurance light illuminates the 'CANCEL' button on the associated Call Button plate and the Call Button may also 'beep' periodically (if activated), providing reassurance to the user. The mat also has its own easily identifiable alert, such as 'Out of Bed Room 1'. This alert is also immediately displayed at the Nurse Station and sent to associated paging devices.

The addition of the Advance Care Foldable Out-of-Bed Floor Mat Interface allows for both the Floor Mat and a Call Pendant to be used simultaneously. The switch on the front of the Interface Device allows for the mat to be deactivated without needing to reposition the mat to avoid false alarms.

This product integrates with most Nurse Call Systems via a 6.35mm pendant plug and is a valued addition to any Advance Care IP Nurse Call installation.

Features

- Non-slip.
- Water-resistant.
- Pressure-sensitive.
- Simple plug-in use.
- Durable construction.
- Reliable out-of-bed detector.
- Suitable with any level of care
 - Neutral in colour
 - Warm underfoot
 - Large size



Code	Description
AC-FFM	Out-of-Bed Foldable Floor Mat
AC-FFM/I	Out-of-Bed Foldable Floor Mat Interface

Design Specifications

Feature	Description
Height	1200mm
Width	600mm
Connectors	6.5mm Jack
Temperature	-20°C to 50°C
Humidity Range	0 - 90%

Compliance

Designed to comply with:

- AS/NZS 3811 Hard Wired Patient Alarm Systems
- AS/NZS 2999 Alarm Systems for the Elderly and Other Persons at Risk
- AS/ACIF S009 Installation Requirements for Customer Cabling
- AS/NZS 3000 Wiring Rules
- AS/NZS CISPR22 (Class B) Electromagnetic Compatibility (EMC)

© 2022 Advance Care™

All rights are reserved.

Advance Care™ reserves the right to make changes in specifications and/or discontinue any product at any time without notice or obligation and will not be liable for any consequences resulting from the use of this publication.

Authorised Distributor

